**Product description:**

**Meron** is a glass ionomer cement for luting.  
**Meron** has a low solubility in the mouth and a low level of acidity.

**Indications:**

For attaching crowns, bridges, inlays, onlays, pins, posts and orthodontic bands.

**Contraindications:**

**Meron** contains polyacrylic acid and parabens. **Meron** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

**Patient target group:**

**Meron** is suitable for use in all patients without any age or gender restrictions.

**Performance features:**

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

**User:**

**Meron** should only be used by a professionally trained dental practitioner.

**Use:**

Thoroughly clean and dry enamel, dentine and cavity surfaces. Clean cores before luting (e.g. pumice, water), thoroughly rinse and dry with a gentle stream of air or cotton pellets (do not overdry!).

Ensure sufficient residual dentine thickness to prevent pulp irritation. If necessary, protect the pulp or build up the core.


**Shake the powder well before use.**

The working temperature should be 15°C - 23°C.

**Use one drop of liquid for one level measuring scoop of powder (powder / liquid ratio = 3.0 : 1 g/g).**

Place powder and liquid on a suitable glass plate or a suitable mixing pad. When dispensing the liquid, hold the dropper bottle vertically.

It is recommended to divide the powder into 3 portions and to mix it portion by portion into the liquid, using a solid plastic spatula or another abrasion-resistant type of spatula.

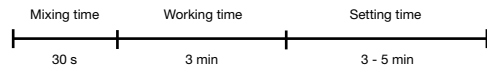**Application:**

**Meron** has a working time of approx. 3 min at room temperature and sets after 3 - 5 min in the mouth. Higher temperatures shorten the working time, low temperatures prolong it.

The correct consistency is achieved when **Meron** can be applied to the core and into the crown with a hard brush. It is essential to prevent contact with water and saliva during application and setting.

Excess material can be removed particularly easily during the rubber-elastic phase (before complete curing) using a scaler or a curette, for example.

**Warnings, precautionary measures:**

- Remaining varnish and other substances (e.g. isolating agent) can affect adhesion and should be removed completely before using **Meron** for final luting.
- Rinse instruments with cold water immediately after use and before setting of the cement residues. After removal of orthodontic bands, cement residues can be removed gently and rapidly with an ultrasonic scaler.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

**Constituents (in descending order according to content):**

Calcium aluminium fluorosilicate glass, water, polyacrylic acid, tartaric acid, parabens

**Storage instructions and application method:**

Reclose powder and liquid containers well immediately after use. Storage at 4°C - 23°C. Do not use after the expiry date.

**Disposal:**

Dispose of the product in accordance with local regulations.

**Reporting obligation:**

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Meron** must be reported to VOCO GmbH and the responsible authority.

**Note:**

The Summary of Safety and Clinical Performance of **Meron** can be found in the European database on medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at [www.voco.dental](http://www.voco.dental).

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