Universal Polishing Paste

[en] Instructions for Use

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1 Intended use

Intended purpose

Polishing of resin or composite veneering, composite components of partial and complete dentures and metal restorations (e.g. Inlays, onlays, crowns etc.).

Intended users

- Dental laboratory technicians (fabrication of restorations in the dental laboratory)

Special training

None

Use

For dental use only.

Description

Universal Polishing Paste is a paste for polishing dental resin, composite and metal materials. In combination with brushes and buffs, the product can be used for prepolishing and high-gloss polishing.

Limitations of use

The product must not be used in the following cases:

- Intraoral use
- Polishing of ceramic restorations

Composition

Water, aluminium oxide, solvent (hydrocarbons C10-C13), ammonium oleate, cocamide diethanolamine, ammonium hydroxide, pigments

2 Application

NOTICE! Observe the Safety Data Sheet (SDS) available at Ivoclar Vivadent AG's website www.ivoclar.com.

NOTICE! Use the following instruments depending on the surface:

Surface	Prepolishing/ polishing	High gloss
Composite surfaces	– Goat hair brush	– Low high gloss: Cotton buff – Intense high gloss: Leather buff
Metal surfaces	– Felt wheels – Hard polishing brushes – Soft polishing brushes	– Cotton buff

1. Apply required amount of product to the surface.
2. Prepolish surface with polishing brush.
3. Polish the surface to a high gloss using a polishing buff.
4. Clean surface with ultrasound in a water bath and/or with the steam jet.
5. Rinse surface with a detergent solution.

3 Safety information

- The product has been developed solely for use in dentistry. Process according to the Instructions for Use.
- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, www.ivoclar.com and your responsible competent authority.
- Check to make sure that the packaging and the product are intact and undamaged before use. If in doubt, contact Ivoclar Vivadent AG or your local sales partner.
- Repeated contact may lead to chapped or cracked skin.

Supporting documents

Document	To be found at:
Current version of Instructions for Use	www.ivoclar.com/elFU
Structure of Instructions for Use and Warnings	www.ivoclar.com/elFU
Explanation of symbols	www.ivoclar.com/elFU
Safety Data Sheet (SDS)	www.ivoclar.com

Disposal information

Dispose contaminated products, remaining stocks or removed restorations according to the corresponding national legal requirements.

4 Shelf life and storage

- Storage temperature 12-28 °C (54-82 °F)

5 Additional information

Keep material out of the reach of children!

The manufacturer accepts no liability for damages resulting from improper use of the product. Furthermore, the user is responsible for determining the product's suitability for the user's own purposes before use.

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UWAGA! Karta charakterystyki (SDS - Safety Data Sheet) dostępna jest na stronie Ivoclar Vivadent AG (www.ivoclar.com).